

Breast Milk Alcohol Test Strip

Feed baby without alcohol, at-Home Test for Breastfeeding Moms



Breast Milk Alcohol Test Strip

Web: <https://breastmilkalcoholteststrip.com/>


E-mail: info@breastmilkalcoholteststrip.com/

Breast Milk Alcohol Test Strip



Oder Code	Package
BMATS-0001	0strips/box


Operation

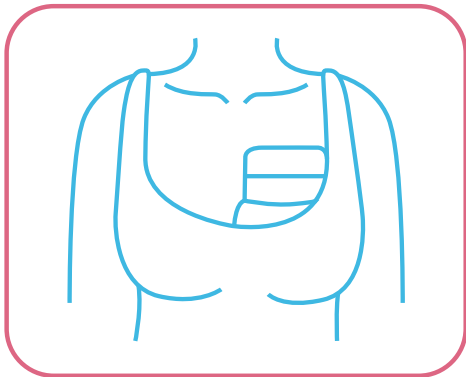


Step 1
Collect the breast milk



Step 2
Dip the strip into breast milk

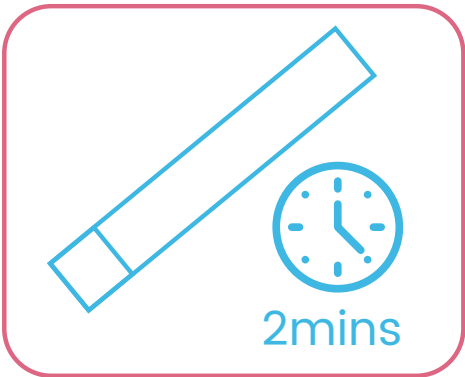

Step 3
Wait for 2 minutes

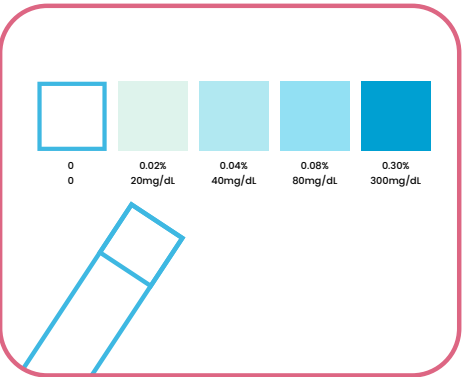

Step 4
Read the result

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
Result interpretation

-



0
0

+



0.02%
20mg/dL



0.04%
40mg/dL



0.08%
80mg/dL



0.30%
300mg/dL

Order

Contact: info@breastmilkalcoholteststrip.com